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Patient Information Leaflet Revamil Wound Dressing

The product
Revamil Wound Dressing <p>10x10 cm honey-impregnated gauze for the treatment of wounds</p>

Product description

Revamil Wound Dressing is a gauze impregnated with pure honey that has been produced under controlled conditions. The carefully controlled production process guarantees that Revamil Wound Dressing does not contain any traces of pesticides and that the same quality can be reproduced consistently. Revamil honey contains a high proportion of honey enzymes, such as glucose oxidase. The highly concentrated sugars cause some of the wound exudate to mix with Revamil gel. The resulting layer helps to strengthen the antibacterial protection provided by the dressing.

Instructions for use

- When required first clean the wound in the conventional way
- Open the packaging at the upper side by tearing open the seal
- Remove the impregnated dressing from the protective sheets and spread it out over the wound surface
- Attach the dressing in the normal way, e.g. by self-adhesive, porous tape
- Revamil Wound Dressing can also be covered by a secondary

- dressing that is attached in the conventional way
- After one to at most three days remove the dressing carefully from the wound and replace it with a new Revamil Wound Dressing

Indications
Revamil wound dressing can be used to treat different types of wounds such as bed sores and different kinds of ulcers, infected wounds, acute wounds, surgical oncological wounds, radiation induced oncological wounds and minor (first- and second-degree) burns.

Contra indications
<ul style="list-style-type: none">Do not use Revamil Wound Dressing if the patient is known to be allergic or hypersensitive to honey In case of unexpected side effects after use contact your doctor or the manufacturer

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Storage conditions
Store between 10 and 30°C

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