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Patient Information Leaflet Revamil Melginate

The Product
Revamil Melginate – Alginate dressing met honing
5x5 cm / 10x10 cm honey-impregnated alginate sheet for the treatment of wounds

Product description
Revamil Melginate is a calcium alginate sheet impregnated with pure honey that has been produced under controlled conditions. The carefully controlled production process guarantees that Revamil Melginate with honey does not contain any traces of pesticides and that the same quality can be reproduced consistently. When Revamil Melginate is applied on exudating wounds a gel is formed by absorption of wound fluid which protects the wound against maceration. Due to the properties of the alginate applied, i.e. the alginate is fast gelling, non-dispersible and has a high wet integrity, the alginate gel covers the wound and maintains a moist wound environment. The alginate sheet impregnated with honey, before and after gelling, protects the wound against bacterial ingress. The gel facilitates easy dressing change because it does not stick to wound and no residues remain in the wound after removal.

Instructions for use

- When required first clean the wound in the conventional way
- Open the outer packaging of the pouch
- Wear disposable gloves when handling the alginate sheet
- Remove the dressing from the transparent blister
- Apply dressing to wound surface
- Cover and secure Revamil Melginate dressing with an appropriate non-occlusive secondary dressing

- In case of high exudate level change the dressing every 24 hours
- In case of moderate exudate level change the dressing every 3 days or earlier when the dressing has reached its absorbance capacity
- If part of the alginate dressing adheres to the wound, moisten the dressing with sterile water or normal saline solution
- Gently discard the alginate dressing from the wound and discard

Indications
Revamil Melginate is intended for the management of different types of exudating wounds, including bed sores, different kinds of ulcers, infected wounds, surgery and radiation induced oncological wounds, minor (1st- and 2nd degree) burns.

Contra indications

- Do not use if the patient is known to be allergic or hypersensitive to honey or alginate
- Do not use on a dry wound
- In case of unexpected side effects after use contact your doctor or the manufacturer

Side effect

- During the first minutes after application on the wound Revamil Melginate impregnated with honey can have a light stinging effect

Safety precautions

- Do not use if the packaging is damaged
- Do not use if the packaging is not sealed tight
- Do not treat a wound for more than 30 days with the product
- Do not reuse an already used dressing. Reuse of a dressing can lead to loss of function of honey, contamination of the product and contamination or infection of the wound
- Do not use after the expiration date
- Do not resterilize an opened packaging

Storage conditions
Store between 10 and 30°C

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