Brugsanvisning Revamil Balm

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Instructions for use
1. Apply a thin layer of Revamil® Balm on the wound
2. The wound can be covered with a wound dressing
3. Treat affected areas once every 24 to 48 hours;
4. In case of persisting itching, Revamil® Balm can be applied more frequently
Indications
Revamil® Balm can be applied on superficial wounds. Revamil® Balm is allowed for all age groups
Contra indications
Do not use Revamil® Balm if the patient is known to be allergic to honey.
Side effects
<ul style="list-style-type: none">During the first minutes after application on minor wounds Revamil® Balm can cause a light stinging sensation In case of unexpected side effects after use contact your doctor or the manufacturer
Safety precautions
<ul style="list-style-type: none">Do not use Revamil® Balm if the tube is damaged Always replace the top on the tube after use Do not allow the tube to come into direct contact with the surface of the wound Once the tube has been opened use the contents within 3 months Do not treat different patients with the same tube
Storage conditions
Store between 4°C and 30 °C

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